# CARE FIX Fixierstrumpf

| Größe | Größenumfang | Artikelnummer      | PZN      |
|-------|--------------|--------------------|----------|
| XL    | 65-95 cm     | 05/02/11/OS/XL/GHC | 12418555 |
| L     | 55-80 cm     | 05/02/11/OS/L/GHC  | 07126017 |
| M     | 45-56 cm     | 05/02/11/OS/M/GHC  | 07126023 |
| S     | 30-45 cm     | 05/02/11/US/S/GHC  | 07126313 |

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## Material:

80% Polyester, 20% Elastan

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### Indikation:

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## Leg Bag Holder

### Indication:

Medical device for securing a urinary leg bag.

### Instructions for use:

1. Pull the CarePocket® over your leg, making sure that the colour marking is at the top.
2. Place the urinary leg bag in the pocket and pull the outlet tap through the lower opening.

3. Your CarePocket® is now properly attached.

### Features:

CarePocket® provides a sleeve for securing a urinary leg bag. The sleeve has a slit at the bottom for the outlet tap of the urinary bag to be pulled through. CarePocket® takes urinary leg bags with a capacity of up to 750 ml/24 oz and is suitable for wearing on the thigh or calf. No external seams

### Precautions:

It is recommended that you empty your urinary bag when half full. If you have any questions concerning use or correct fitting of your CarePocket®, or if you notice any unexpected discomfort or skin irritation, you should consult your healthcare professional.

For device complaints, please contact your supplier. Please also report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority at the user's place of residence.

### Care instructions:

For optimum durability, it is recommended that you wash the leg bag holder in a laundry bag at 60°C, then dry in the tumble dryer on a gentle setting. Chlorine-based bleaches and disinfectants will shorten the life of the product. Please keep these instructions in a safe place for future use.

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