



Instruction for Use

Product Name

Reusable Incontinence Pad

Model

40x50cm, 45x45cm, 45x60cm, 50x50cm, 50x60cm, 51x56cm, 53x56cm, 45X61cm, 75x90cm, 76x86cm, 80x90cm, 85x90cm, 85x135cm, 86x91cm, 86x132cm, 86x138cm, 86x96cm, 90x132cm, 90x140cm, 90x120cm, 90x150cm, 90x180cm, 91x91cm, 91x138cm, 91x142cm, 91x152cm, 91x182cm, 100x140cm, 110x130cm, 111x132cm All the size can be produced with or without tucks/wings

Device Description

A flat piece of absorbent fabric material [e.g. polyester+ non-woven polyester with viscose + polyester with PU/PVC, or polyester + non-woven polyester with viscose + polyester with PU/PVC + cotton/polyester fabric, or polyester + padding + non-woven polyester with viscose + polyester with PU] intended to be placed on a surface (e.g., bed mattress, chair, sofa) and underneath a patient to absorb bodily fluids resulting from surgical procedures, childbirth and/or incontinence; it does not include an antimicrobial agent(s). It is intended to be used in the healthcare facility and in the home.

Intended Use

The intended use of this product is for use in homes, hospitals, and nursing homes to protect the elderly and incontinent patients from urine or other body fluids seeping into beds, sofas, chairs, or other areas and to provide isolation and protection.

Appearance and Components

Made from warp knit fabric, non-woven polyester, padding polyester and TPU/PVC composite or sewn. Provide non-sterile.

Usage

1. Read the label and instruction for use carefully before used;
2. Take off the package, do not use the product once the package was damaged or the product has expired;
3. Put the incontinence pads on the sick bed/ chair/sofa or other surfaces that need to be protected and isolated.

Precaution and Warning

1. This product is reusable, please timely change the clean product.
2. This product is not sterilized before delivery and is not provided aseptically.
3. Do not use it with electric blankets and other heating appliances.
4. Please keep the product out of reach of children and old people to avoid accidental ingestion.

Contraindications

Do not use if you are allergic to warp knit fabric

Storage

These products should be stored in a cool dry area, away from heat and direct sunlight.


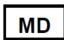








Cleaning Method








1. Do not use fabric softener wash before initial use
2. Washing temperature < 95°
3. Tumble dry at low temperature
4. Do not Dry Clean/ Do not Bleach/ Do not Iron/Do not use chemical cleaning
5. Do not steam drying or press drying

Shelf Life

Five years

Labels, Packing Logo Design

Symbol	Introductions	Symbol	Introductions
	CE Symbol		Medical device
	Manufacturer name and address		Authorised representative in the European Community
	Manufacture date		Use until year & month (expiration date)
	Batch code		Do not reuse / single use
	Warnings and precautions		Non-sterile

	Consult instructions for use		Do not use if package is damaged
	Unique device identification		Country of manufacture
	Importer		Distributor
	Model number		

Manufacturer Information



Company: Changzhou Newrise Medical & Hygiene Products Co., Ltd
Address: No. 39 Chuangye Road, Zhulin Town, Jintan District,
Changzhou City, Jiangsu, China 213241

European Authorized Representative (EU REP)



Company: SUNGO Europe B.V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands