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EN Transurethral balloon catheter

General information

Dear user, please read all of the following information carefully before using the transurethral balloon catheter.

Note

We wish to point out that the products above may be used solely by physicians and trained specialists upon medical referral.

Uses/Description

A transurethral balloon catheter is used for the treatment of bladder emptying disorders as well as for permanent bladder drainage.

Storage

- This product should be kept in its original packaging.
- Do not keep close to direct or indirect sources of light or heat. Store in a dry place.
- Keep out of the reach of children.

Packaging/Presentation form

- Each article comes in sterile, individual packaging. Sterility is only ensured for as long as the packaging is unopened and undamaged, but for no longer than the date specified on the label.

Indication

- For bladder emptying problems with pathological residual urine or urinary retention.
- For instances where urine output needs to be balanced and for monitoring kidney function.
- Urine incontinence etc.

Contraindications and restrictions on use

The catheter may not be used, or may only be used if there is a definite indication, in the following instances:

- Urinary tract infection (cystitis/urethritis)
- Injury to the urethra/obstruction of the urethra (through CA, for example)
- Refusal on the part of the patient or his/her carer
- Urethral stricture
- Injury to the penis

When using a latex catheter, a latex allergy test must be performed beforehand.

Instructions for use/Procedure

Preparing the patient:

- Both the preparation of the patient and the catheterisation procedure must be done using the relevant techniques, as can be found in the specialist medical literature, and under sterile conditions.
- Gender-specific disinfection is to be carried out in accordance with the specialist medical literature
- Male patients: application of lubricant into the urethral orifice; allow to take effect. Only use medical lubricant authorised for this purpose.

Procedure

- When using the transurethral 2-way balloon catheter, it must be ensured that the Tiemann tip is pointed upwards.
- Carefully insert the catheter into the urethra
- As soon as urine starts to flow, insert the catheter a further 3-4 cm into the urethra
- Block the balloon with 10 % glycerol solution (observe filling volume) and pull back towards the bladder wall
- Conned the catheter to a suitable urine drainage system (e.g. GHC UB 2000).

Removing the catheter

- To unblock the catheter, connect a Luer lock syringe to the valve. Slowly draw the syringe up in order to remove the fluid from the balloon. Do not draw the syringe plunger back too quickly, as this can lead to under-inflation, thereby causing the filling chamber to collapse and preventing normal balloon drainage. Note: make sure that the balloon is drained completely.
- Should the catheter not be able to be unblocked, as can happen in rare instances, please contact the physician in charge.

Disposal

- Dispose of the catheter and its packaging in accordance with the national regulations in force.

Indwelling periods

- According to the recommendation made by the Robert Koch Institute, „indwelling bladder catheters should not be routinely changed at fixed intervals, but as required, according to individual requirements (e.g. incrustation, obstruction, soiling).“
- Balloon catheters suitable for long-term drainage:
 - Transurethral silicone-elastomer 2-way balloon catheter
 - Transurethral 100 % silicone 2-way balloon catheter

- Balloon catheters suitable for short-term drainage:
 - Transurethral latex (siliconised) 2-way balloon catheter

Warnings/Precautions

Never insert the catheter against resistance. In the event of resistance, discontinue the catheterisation procedure and contact the physician in charge. Never block the balloon against resistance. If, in rare cases, the catheter is not able to be unblocked even after inserting the Luer syringe for up into the valve, please contact the physician in charge. Do not use silicone-containing sprays as lubricant when using silicone catheters. Do not use petroleum-based lubricants on products made of latex. Block the balloon with 10 % glycerol solution. Never clamp the catheter; if necessary, use a catheter valve or stopper. Never puncture the catheter shaft with a syringe to drain off a urine sample. This may compromise or damage the catheter.

Disclaimer

No liability is assumed for any injuries or other damage resulting from improper use of the product.

ATTENTION: All information has been provided to the best of our knowledge at the time of going to print. Subject to technical changes.

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Haiyan Kangyuan Medical Instrument Co. Ltd. Songpoding Rd. Shendang Town 314311, Haiyan, Zhejiang China
Eunitor GmbH Kennedydamm 5 40476 Düsseldorf Germany
Obelis Swiss GmbH Ruessenstrasse 12 6340 Baar/ZG Switzerland