
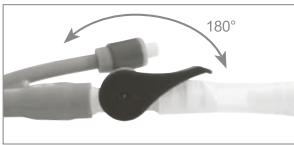
CARE FLOW Katheterventil Artikelnummer: 03/02/10/00/GHC

GHC German Health Care GmbH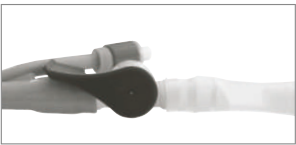

Geschlossene Position des Katheterventils (siehe roter Pfeil – close) | Closed position of the catheter valve (see red arrow - close) | Valve de cathéter en position fermée (voir flèche rouge - close) | Posición cerrada de la válvula de catéter (ver flecha roja – close) | Valvola per catetere in posizione chiusa (vedere freccia rossa – close) | Gesloten positie van het katheterventiel (zie rode pijl – close) | Lukket position for katheterventilen (se rød pil – close)

Offene Position des Katheterventils / Dauerableitung (siehe grüner Pfeil – open) | Open position of the catheter valve / permanent drain (see green arrow - open) | Valve de cathéter en position ouverte/évacuation continue (voir flèche verte – open) | Posición abierta de la válvula de catéter / derivación continua (ver flecha verde – open) | Valvola per catetere in posizione aperta / drenaggio permanente (vedere freccia verde – open) | Open positie van het katheterventiel / continue afvoer (zie groene pijl – open) | Åben position for katheterventilen/permanent afledning (se grøn pil – open)

CARE FLOW Katheterventil

D

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GB

Instructions for use

A discreet, comfortable, easy to operate catheter valve with a safety lever feature to prevent accidental opening and a universal stepped connector which can be inserted into all catheter materials.

Introduction

We understand the importance of your independence and the catheter valve is especially designed to ensure you feel comfortable and confident every day.

Our catheter valve has been prescribed for you by a health care professional following a bladder and catheter assessment. It is important that you use the valve as instructed by your doctor or nurse as part of your ongoing catheter management plan.

Our catheter valve is a medical device designed to be used with an indwelling urethral or suprapubic catheter. It is suitable for men and women.

Only use our catheter if you (or your carer) have the ability to operate it at regular intervals and can remember to do so.

Benefits of using a valve

Our catheter valve is a discreet alternative to urine drainage bags, however unlike bags it enables intermittent bladder filling and drainage, therefore maintaining normal bladder function.

Contraindications

Our catheter valve may not be suitable for individuals who:

- Are unable to operate the valve
- Are unable to remember to operate the valve
- Have a small bladder capacity
- Have reduced bladder sensation
- Have renal impairment or ureteric reflux
- Have uncontrolled overactive bladder

How to drain your bladder using catheter valve

Drain your bladder using the valve as frequently as recommended by your doctor or nurse:

1. Wash and dry your hands
2. Hold the valve over a toilet or suitable receptacle
3. Open the valve by pushing the lever tap fully into the downward position (you may feel some mild tension in the lever, this is normal – it is the safety feature working to prevent accidental opening)
4. Allow the valve to drain the bladder completely (the flow may be slow – this is due to the narrow lumen inside the catheter)
5. Once urine has stopped draining, close the valve by lifting the lever tap fully into the upright position
6. Dry the end of valve with a clean wipe if necessary
7. Wash and dry your hands

Change your valve every 7 days or sooner if damaged or contaminated.

If you use a drainage bag overnight then this can be connected to the soft silicone outlet tube on the end of catheter valve. Ensure the lever tap is in the open position to allow continuous drainage.

Your catheter valve is supplied sterile. It is important that it is only connected once then discarded and no attempt is made of re-sterilise or reuse the valve.

If the valve packaging is broken, do not use the valve as it will no longer be sterile.

If you are concerned that your catheter is not draining urine and you are in discomfort then contact your doctor or nurse immediately.

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			
Conod Medical Co., Limited. No. 11 Hongfeng Road, Baimao Industrial Park, Guli Town, 215532 Changshu City, Jiangsu Province, People's Republic of China	Shanghai International Holding Corp. GmbH (Europe) Eiffestraße 80, 20537 Hamburg, Germany	The Tao of Excellence GmbH Vorstadt 26, CH-8200 Schaffhausen, Switzerland	GHC German Health Care GmbH Birklick 15, 24999 Wees, Germany Tel. +49(0) 4631 / 572 00 0 info@ghc-medical.de www.ghc-medical.de
	0123		
Rev.006/07.2024			

Bei beschädigter Verpackung nicht verwenden / Do not use if package is damaged / Ne pas utiliser si l'emballage est endommagé / No utilizar si el envase está dañado / Non utilizzare se l'imballaggio non è integro / Bij beschadigde verpakking niet gebruiken / Må ikke anvendes ved beskadiget emballage

Vor Sonnenlicht schützen / Keep away from sunlight / Conserver à l'abri de la lumière du soleil / Manténgase fuera de la luz del sol / Tenere lontano dalla luce / Beschermen tegen zonlicht / Beskyttes mod sollys

Trocken aufbewahren / Keep dry / Craint l'humidité / Manténgase seco / Mantenere asciutto / Droog bewaren / Opbevaes tørt

Nicht wiederverwenden / Do not re-use / Ne pas réutiliser / No reutilizar / Non riutilizzare / Niet opnieuw gebruiken / Må ikke genanvendes

Gebrauchsanweisung beachten / Consult instructions for use / Consulter les instructions d'utilisation / Consúltense las instrucciones de uso / Consultare le istruzioni per l'uso / Gebruiksaanwijzing raadplegen / Overhold brugsanvisningen